

# Compass

## Access Assessment Software

### What is Compass Software?

Compass software takes the guesswork out of computer access assessment. It provides clear evidence to show which input device and display settings best meet the needs of the end user.

### How does Compass work?

Compass provides you with eight skill tests in three categories (pointing, text entry, and switch use). You choose the appropriate tests to run, based on the skills you want to assess. Compass records and reports the speed and accuracy of the user's actions.

This information can lead to better computer access solutions. For example, Compass helped one team pinpoint and solve a user's keyboarding difficulties, enhancing typing speed by 50%!

### Who will benefit from Compass?


End users benefit from access solutions that are based on evidence. Compass helps identify and quantify user needs, and documents when a successful solution has been achieved.

Assistive technology teams benefit by using a tool that quickly and effectively gathers evidence to aid decision-making. Compass makes it easy for teams to report and share information, too.

**Compatible with Windows & Mac OS X**

Developed by:

 **KPR** Koester Performance Research  
www.kpronline.com

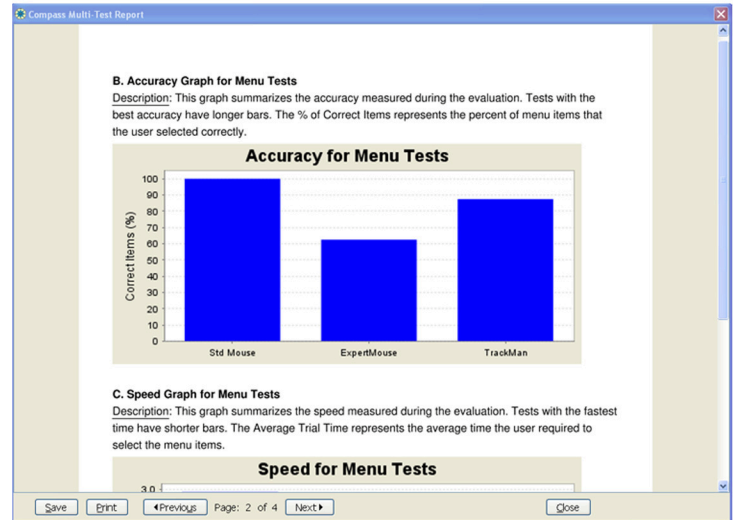

**Summary Results: Aim 1**

Client: Susan Jones  
Test Date: Feb 24, 2011 12:18:50 PM  
Evaluator:

**Test Configuration**

Setting	Value
Input Device	Not Specified
Test Language	English
Number of Trials	12
Selection Method	Click
Target Type	Squares
Target Size	Icon
Target Distance	Medium Long
Draw Path	No
Target Color	Blue
Background Color	White
Screen Coverage	100%
Maximum Time	30 sec
Pause Time	1 sec
Feedback Style	Basic

**Description of the Aim test**  
The Aim test asks the user to acquire targets with a pointing device. The user moves the cursor to each target and selects it.

**Summary of Results**

Result	% (N)	Trial Time (sec)	Reaction Time (sec)	Entries	Clicks
Error-free Target Selections	100% (12/12)	0.88	0.13	1.08	1
All Targets Selected	100% (12/12)	0.88	0.13	1.08	1
Targets Not Selected	0% (0/12)				
All Aim Trials	100% (12/12)	0.88	0.13	1.08	1

**Target Map**

Green circles indicate the position of targets which were successfully selected. Red X's indicate the position of targets that were not selected in the available time.

Buttons: Print Report, View Details, Save & Continue, Cancel

**View Demonstration Video:**  
[www.kpronline.com/compass-demo.php](http://www.kpronline.com/compass-demo.php)

**For a Free Trial:**  
Call 800-397-0921 or visit [kpronline.com](http://kpronline.com)

Distributed by:

 **infogrip**  
www.infogrip.com